

LIGHTLas SLT Deux-V

SLT/YAG COMBINATION SYSTEM



CLINICAL VERSATILITY WITH
CUTTING EDGE TECHNOLOGY



ADVANCED DESIGN PROVIDES INNOVATIVE THERAPY



Proven effective in treating secondary cataract, glaucoma and vitreous floaters, with expanding applications in retinal diseases, the LIGHTLas SLT Deux-V offers industry-leading versatility.

Thermal KTP Crystal Management

Exclusive Circuitry and Software

Eliminates KTP variation by engaging a unique thermal control mechanism:

- Guarantees optimum, shot-to-shot performance and reduces energy variation errors.
- Assures most stable SLT output energy performance over high repetition firing.

Crystal Q-Switch Laser Technology

Powerfully Crafted

The unit is operated through a laser fire Q-Switch conveniently integrated into the system.


Unique Laser Cavity Technology

Provides optimum tissue-cutting precision and consistent shot-to-shot output energy at the industry's lowest optimal breakdown levels.


Advanced Noise Reduction

Improves patient compliance and allows procedures to be completed with lower energy levels to help reduce treatment side effects and lens pitting.

LIGHTLas SLT Deux-V Energy Stability

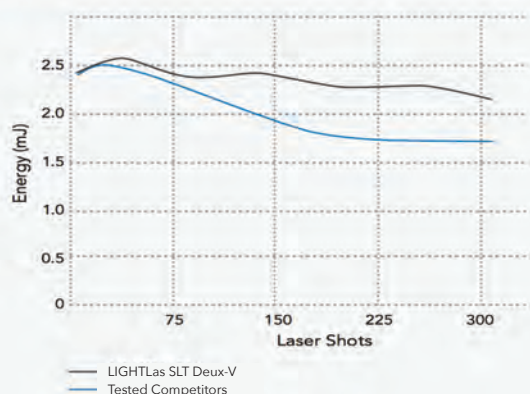

LIGHTLas YAG-V Crystal Q-Switch Gaussian Profile

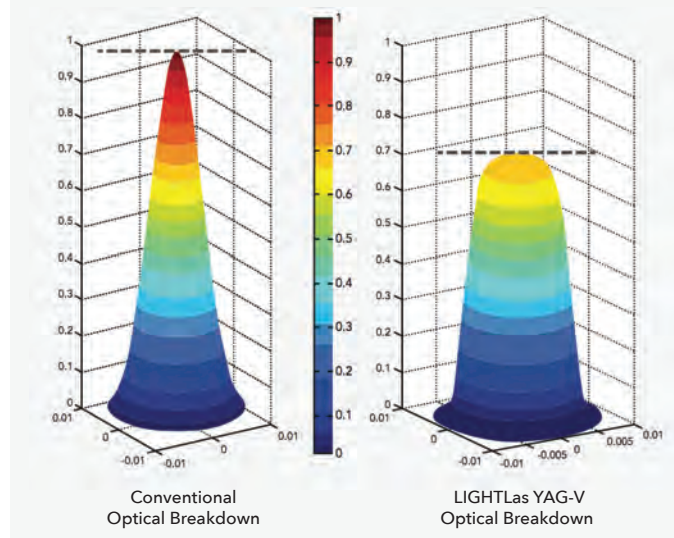

“LIGHTMED provided a YAG/SLT laser for me to evaluate. The optics are terrific, the laser energy is precise and everything about the console feels like a quality product. If you are in the market for an ophthalmic, office-based laser, do yourself a favor and get LIGHTMED to show you their offerings, you will be happy you did.”

Toby Tyson, MD; Cape Coral, FL

FULL ARRAY OF FEATURES AND BENEFITS

Intelligent and ergonomic features offer superb precision and unparalleled ease of use, ensuring optimum clinical outcomes and unmatched value.

Quality Precision Products

Superior Design

High-resolution, LED slit lamp along with quality components provide an enhanced view and seamless operation.

Large, Crisp Field of View


Beam splitter-free design features internally coated safety optics to maximize resolution and viewing clarity.

Five-Step Magnification Changer

Provides exceptional viewing, from fine structures to the wide-field view of the retina. The integrated magnification changer helps improve diagnosis capabilities at a convenient working distance.



LIGHTLas SLT Deux-V
Field of View

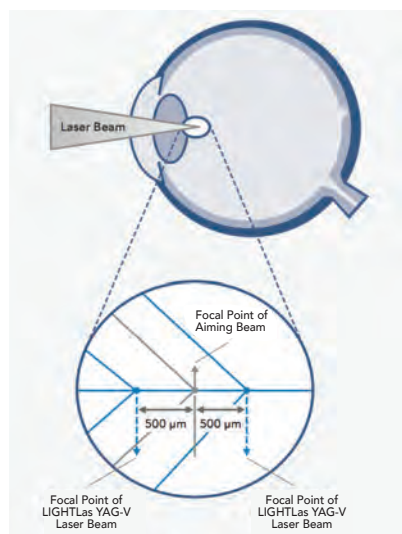

LIGHTLas YAG-V
 $\pm 500 \mu\text{m}$ Posterior/
Anterior Offset

Posterior/Anterior Offset $\pm 500 \mu\text{m}$

Large Focus Shift

Allows detailed titration of treatment focus without compromising comfort, and preventing the possibility of lens pitting.

Clinical Versatility

Essential for multi-patient environment with numerous IOL types.

Adjustable Setting

Extensive offset range of $\pm 500 \mu\text{m}$ for higher accuracy and greater control.

SEAMLESS DUAL FUNCTIONALITY: YAG & SLT IN ONE PLATFORM


Effortlessly switch between YAG and SLT treatments with a single platform - saving space, time, and simplifying your workflow.

Advanced Engineering

- The LIGHTLas SLT Deux-V design simplifies service and maintenance, resulting in minimum system downtime and reliable operations.
- Innovative engineering enhances practice dynamics and clinical scope.


Unmatched Ergonomics

- **Convenient Operation:** Dual hand controls enable ease of use while externally mounted chin rest facilitates patient posture and comfort.
- **Modular Design:** Assures better treatment and offers faster and easier maintenance.
- **Perfect Precision:** Advanced laser firing mechanism utilizes a direct fire-to-joystick switch mechanism for optimal performance.

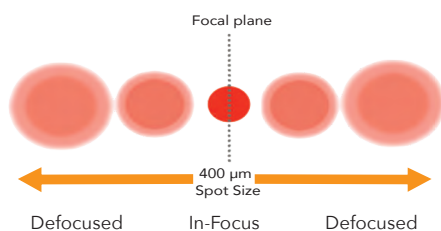

SLT Aiming Focusing System

Single SLT aiming beam focusing is easily achievable at focal plane, producing crystal clear spot with visible internal inner rings.

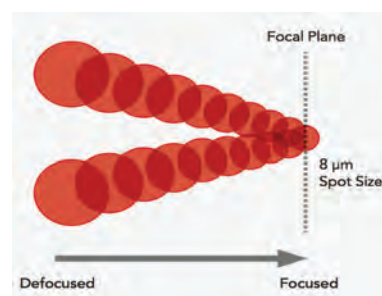

YAG Twin Aiming Focusing System

YAG twin aiming focusing system allows both beams to converge together at the focal target to create a sharp and precise spot.

“ I've worked with different devices for 15 years prior to acquiring the LIGHTLas YAG/SLT Deux-V, and visibility is way better. The optics are so superior, and with the gonio lens, everything is clear especially when finding the trabecular meshwork. Essentially, you have one machine that fits all offices, offering three different approaches to glaucoma: SLT, SPLT, YAG PI. You can also add the retina laser with a simple adaptor that includes SP-Mode® Microsecond Laser Technology. ”

Nir Shoham-Hazon, MD; Miramichi, New Brunswick, Canada

DYNAMIC WITH INFINITE TREATMENT OPTIONS


In addition to a suite of advanced features, LIGHTLas SLT Deux-V offers a comprehensive selection of combinations as your practice grows and clinical needs change.

Why Selective Laser Trabeculoplasty (SLT)?

Selective Laser Trabeculoplasty is a safe and effective laser procedure offered as the standard of care in the treatment of Primary Open Angle Glaucoma (POAG). The treatment is indicated to lower the Intra-Ocular Pressure (IOP) associated with glaucoma.

- A concentrated low energy laser beam is directed at the trabecular meshwork with an SLT lens.
- The short pulses of green (532nm) wavelength and low energy of the laser light stimulates the body's natural immune response to rebuild the functions that associate with regulation of the IOP.
- Over several weeks this response promotes improved fluid flow from the eye, resulting in lower IOP. There is minimal thermal damage which could lead to less scar tissue formation because the laser energy only interacts with pigmented cells.
- SLT is proven to be safe, and can be repeated without any known side effects.

Selective Laser Trabeculoplasty (SLT)

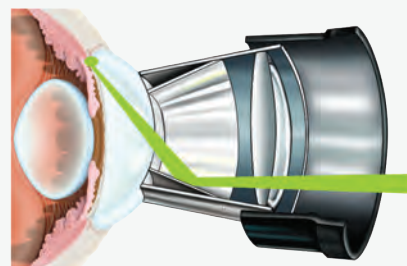

Range of Workstation Options

- **Powerful Multi-Functional Integration:**
 - Upgrade with the LIGHTLas 532 (green), LIGHTLas 577 (yellow), or LIGHTLas 810 (infrared) with a slit lamp adaptor to expand clinical scope.
 - LIGHTLas photocoagulators utilize continuous wave (CW) and exclusive SP-Mode® Microsecond Laser Technology delivered through the slit lamp of the LIGHTLas SLT Deux-V.
- **Treatment Adaptability:**
 - LIGHTLas photocoagulators can be conveniently located on a SMART medical cart, or placed on your table of choice for a complete mobile workstation.

LIGHTLas SLT DEUX-V SPECIFICATIONS

Mode	YAG Mode	SLT Mode
Laser Type	Q-Switched Nd:YAG	Q-Switched frequency doubled Nd:YAG
Wavelength	1064 nm	532 nm
Energy Range	0.2 to ≤ 15 mJ (in single pulse mode), 10 to ≤ 25 mJ (in double pulse mode), 20 to ≤ 45 mJ (in triple pulse mode)	0.2 - 2.6 mJ, continuously variable
Pulse Width	4 ns	3 ns
Burst Mode	1, 2, and 3 pulse per shot, selectable	Single pulse
Mode Structure	Fundamental, diffraction limited	Frequency-doubled, diffraction limited
Average Air Breakdown	2.1 mJ (≤ 1.5 mJ in liquid solution)	N/A
Spot Size	8 μm	400 μm
Cone Angle	16 degrees	< 3 degrees
Treatment Beam Offset Range	±500 μm, continuously variable	N/A
Aiming Beam	Dual beam laser diode, continuously variable, red 635 nm	Single beam laser diode, continuously variable, red 635 nm
Slit Lamp Illumination LED	Slit lamp XLamp® XM-L2 2.85 V 10 W	
Laser Repetition Rate	Up to 3.0 Hz	
Laser Delivery	Galilean Slit Lamp integrated, stereoscopic 16x microscope with converging optics	
Magnification	Integrated: 5-position, 5x, 8x, 14x, 25x, 38x	
Safety Filter	Fixed OD5 @ 1064 nm and 532 nm, double coated / color balanced	
Cooling	Air convection, passive	
Power Requirements	100-240 VAC, 50-60 Hz, Auto ranging	
Power Rating	200 VA	
Dimensions	45cm (L) x 34cm (W) x 52cm (H) 18in (L) x 13in (W) x 20in (H)	
Weight	24kg, 53lbs (system); 33kg, 73lbs (packed)	

Specifications are subject to change without notice. LIGHTMED devices are made strictly in accordance with the international laser safety regulations and standards: IEC/EN 60601-1, IEC/EN 60601-1-2, IEC/EN 60601-2-22, IEC/EN 60825-1

LIGHTLas SLT is also available upon request (SLT only without YAG)

Optional Accessories

- Dual plug beam splitter
 - Observation tube
 - Photographic camera adaptor
 - Video camera adaptor
- Iridotomy laser lens
 - Capsulotomy laser lens
 - SLT laser lens

Tables

- Single column u-recessed table

*FDA and CE registered model name: Lightlas SeLecTor Deux





LIGHTMED CORPORATION

No. 1-1, Lane 1, Sec. 3, Pao An St. | Shulin District | New Taipei City 23861 | Taiwan

T: +886 2 2688 1726 | F: +886 2 2676 4920

1130 Calle Cordillera | San Clemente, CA 92673 | USA

T: +1 949-218-9555 | F: +1 949-218-9556

www.lightmed.com | sales@lightmed.com